

A-SCAN AND IOL CALCULATION

Ultrasound biometry is the only technology that allows measurements in all eyes types including when dense cataract are present. **With the immersion technique, the axial length measurement accuracy is 0.03mm.**

The **patented A-scan Probeam™** technology is an exclusivity from Quantel Medical. This probe generates a laser beam that offers a fixation point to the patient facilitating: this facilitates the measurement while increasing ease of acquisition.

The **IOL calculation** function allows comparison between different IOL types and calculation formulae.

A total of 12 calculation formulae is available including post-refractive surgery formulae.
The **IOL implant calculation is done at 0.25D.**

SRK-T

HOLLADAY

HOFFER-Q

HAIGIS

SRK 2

BINKHORST - II



PACHYMETRY

Essential for glaucoma diagnosis and for refractive surgery, Compact Touch offers several measuring modes and corneal maps with an accuracy of ± 5 microns.

Intraocular pressure can be adjusted thanks to built-in IOP correction calculation tables.



B MODE

Grey levels: 256
Adjustable gain: 20 to 110 dB
Time Gain Control (TGC): 0 to 30 dB
Dynamic range adjustment: 25 to 90 dB
Storage of still images and video sequences (up to 40 second duration) on hard disk
Image post-processing tools: calipers, areas, markers, comment

15 MHz probe

Transducer frequency: 15 MHz
Angle of exploration: 50°
Depth of exploration: 60 mm (2.36")
Focus: 24 mm (0.94")
Axial resolution: 115 μ m
Lateral resolution: 400 μ m
Frame Rate Acquisition: up to 16 Hz

BIOMETRY

Adjustable gain: 20 to 110 dB
Time Gain Control (TGC): 0 to 30 dB

11 MHz Probe

Transducer frequency: 11 MHz
Tip diameter: 7 mm (0.28")
Electronic resolution: 0.03 mm (0.002")
Depth: 60 mm (2.4") on 1536 points
Contact and immersion techniques compatible
Aiming beam: LED or laser pointer ProBeam™*

Axial length measurements

Ultrasound propagation velocity adjustable per segment (anterior chamber, lens, vitreous) and IOL and vitreous material
Built-in pattern recognition: phakic, aphakic, PMMA, acrylic and silicone material for pseudo-phakic eye types
Automatic calculation of standard deviation and average total length (series of 10 measurements)
Acquisition modes: automatic, auto + save, manual
Automatic detection of scleral spike

IOL calculation

SRK-T, SRK 2, HOLLADAY, BINKHORST-II, HOFFER-Q, HAIGIS
Post-op refractive calculation:
- Pre-op and Post-op refraction, Pre-op and Post-op keratometry
- 6 different methods for keratometric correction and implant calculation:
History derived, refraction derived, contact lens method, Rosa regression, Shamma regression, Double K/SRK-T (Dr. Aramberri's formula)
9 values bracketed for desired ametropia for each IOL (IOL increment steps: 0.25D or 0.50D)
Simultaneous display of 4 different IOL calculations

DATA MANAGEMENT

Built-in physician and patient database
Exportation of still images and video sequences
Customizable digital and printed reports
DICOM compatible (Worklist, Storage, Print)*
EMR compatible
Compatible with PC and USB video printers

Compact Touch®

TECHNICAL SPECIFICATIONS

PACHYMETRY*

Transducer frequency: 20 MHz
Tip diameter: 1.2 mm (0.05")
Method: contact
Convergence: 0.5 mm (0.02") from the tip
Angle: 45°

Corneal thickness measurements

Measurement range: 200 to 999 microns
Number of measurements: 1 to 10
Precision: ± 5 microns
Velocity: adjustable
Methods: central measurement or cartographic map (automatic, continuous, scanning)
user - 8L - 4L - 9C8L - 9C4L - 5C8L - 5C4L - 9C - 5C

I.O.P. correlation tables

Tables correlating intraocular eye pressure and corneal thickness:
Ehlers + Doughty + Dresdner + unlimited user-defined tables

Specifications

Bias correction: up to 120%

GENERAL INFORMATION

Back-lit LCD colour touch screen monitor (resolution 1024x768)

Electrical requirements

Power supply: 100-240 Vac $\pm 10\%$ single phase without earth
Frequency: 50/60 Hz
Power: 60 W max

Features

Overall dimensions: 26.8 cm (W) x 4.0 cm (D) x 24.6 cm (H)
10.6" x (W) x 1.6" (D) x 9.7" (H)
21 cm (W) x 16 cm (H) - 8.3" (W) x 6.3" (H)
Weight: 3.5 kg (7.7lb)
Ports: 4 USB, 1 ethernet

Peripherals and accessories included in the basic configuration

Footswitch
Bluetooth mouse

Peripherals and accessories in option*

Keyboard with USB and bluetooth
Mouse with USB
External PC printer Windows Operating System compatible (USB or Wifi)
Video printer with USB connection

(*) Option

Specifications are subject to change without notice.

©2017. Quantel Medical, Compact Touch® and ProBeam™ are trademarks of Quantel Medical.
All rights reserved.

www.quantel-medical.com

A product by Quantel Medical, France

Headquarters

Quantel Medical
11, rue du Bois Joli - CS40015
63808 Courmon d'Auvergne - FRANCE
Tel: +33(0)4 73 745 745
Fax: +33 (0)4 73 745 700
E-mail: contact@quantel-medical.fr

North America

Quantel USA
49 Willow Peak Dr.
Bozeman, MT 59718 - USA
Tel: +1 877 782 6835
Fax: +1 406 522 2005
E-mail: info@quantelmedical.com

Representative Offices

Thailand, Chiang Mai
Brazil, Rio De Janeiro



Artwork by www.orencart.fr

Photo credit: Fotolia
XE_CTT_B08_JAN_0717

Compact Touch®



COMPACT IN
DESIGN,
DEFINITIVE IN
DIAGNOSIS

A/B/P Ultrasound Platform



Compact Touch

Heir of a **brilliant** past, turned toward **future**


■ QUANTEL MEDICAL REDEFINES THE STANDARD IN B-SCAN IMAGING

The new Compact Touch benefits from a **new generation of 15 MHz B probe** with a **resolution increased by 30%**. It allows for a better visualization of the eye structures and the orbit hence a better diagnosis.

Small in size, this probe benefits from an **excellent ergonomics** to facilitate its handling and use.




© Peter Good, MD
Birmingham and Midland Eye Centre (Birmingham, UK)
Choroidal melanoma

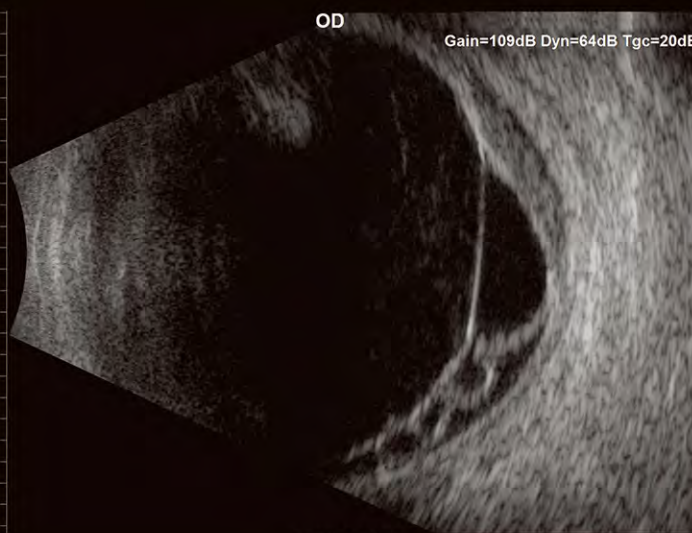

© Adil El Maftouhi - Hôpital des XV-XX (Paris, France)
Centre Ophtalmologique Rabelais (Lyon, France)
Diabetic Retinal Detachment

The Compact Touch has the exclusive technology of **biometry in B-mode** that allows to automatically measure the axial length from a B-mode image.

This technique is essential for the patients with long myopic eyes associated with staphylomas.

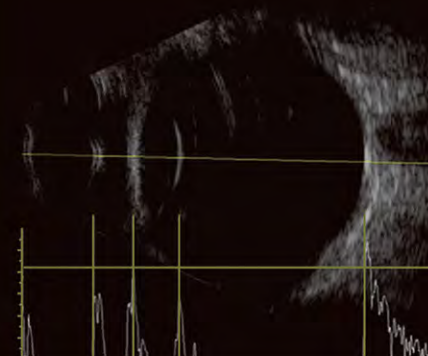

■ A CONNECTED ULTRASOUND PLATFORM

- With a **DICOM interface**, the new Compact Touch can now import (worklist function) and export (storage function) images and patients reports to the PACS. Reports and images printout is also possible either on a **DICOM printer**, or a **local printer with WIFI**.
- **Videos sequences (CINELOOPS)** can also be sent in **DICOM format**.
- For more ease of use, a wireless keyboard and mouse can also be connected.
- A new **HDMI** video output.



■ A SLEEK DESIGN, WITH INCREASED ERGONOMICS

A weight now **less than 4kg**, Compact Touch has never been so well named, since its dimensions have been significantly reduced to fit better in every working environment.

- It is more easily transportable, thanks to its foldable and **reclining carrying handle**.
- To further increase working ergonomics around the patient, its **VESA fixation system** offers the possibility to be **mounted either on a wall**, on an **articulated arm**, or on a **mobile trolley** (optional).
- Fanless, it is as **silent** as a tablet.



ARTICULATING ARM ATTACHED TO WALL OR TABLE*

Non-contractual picture



COMPACT TOUCH ON CART*

Non-contractual picture

